

How to speak with your child's healthcare professional about EndeavorRx™ treatment.

If your child is struggling with paying attention and focus, consider using this guide to start a conversation with your child's healthcare professional about EndeavorRx, a new type of attention treatment for children with ADHD.

You may want to bring this guide with you to your next appointment. And, to get a head start, fill out and print the prescription enrollment form and share it with your child's healthcare professional (pediatrician or psychiatrist).

WHAT IS ENDEAVORRX?

- The EndeavorRx app is a digital treatment indicated to improve attention function as measured by computer-based testing in children ages 8-12 years old with primarily inattentive or combined-type ADHD who have a demonstrated attention issue.
- EndeavorRx is an FDA-cleared medical device and must be prescribed by a healthcare professional.



HOW DOES IT WORK?

- The EndeavorRx treatment is delivered through an immersive and challenging video game experience.
- EndeavorRx works specifically by using sensory stimuli and simultaneous motor challenges designed to target the area of the brain that plays a key role in attention function.
- As a child progresses in the video game, the technology is continuously measuring their performance by using adaptive algorithms to adjust the difficulty and personalize the treatment experience for each individual.



HOW WILL IT BE PRESCRIBED?

- A prescription for EndeavorRx provides an initial 3-months of access to the treatment.
- Upon filling the prescription, your child should play EndeavorRx for approximately 25 minutes a day, for at least 5 days a week over at least 4 consecutive weeks, or as recommended by your child's healthcare professional. At the end of the 3-month access period, your child's healthcare professional should be consulted to determine if your child should continue treatment.

Questions? Our Akili Assist Representatives are here to help.

Give us a call at 1-844-AKILI-IQ (1-844-254-5447)

Mon to Fri: 9 am - 8 pm ET



AkiliAssist.com

Fax number: 1-866-565-4633

PO Box 1517, Woodstock, GA 30188

HOW SHOULD IT BE USED?

- EndeavorRx should be considered for use as part of a therapeutic program that may include clinician-directed therapy, medication, and/or educational programs, which further address symptoms of ADHD.
- EndeavorRx is not intended to be used as a stand-alone therapeutic and is not a substitution for your child's ADHD medication.



WHAT IS TREATMENT LIKE?

The EndeavorRx treatment might feel repetitive and challenging for some children, and it takes commitment from you and your child to see results. You should expect tough moments on the path to improving your child's attention, but making this part of your family's routine can help.

Full details about EndeavorRx, including clinical trial data and important indications, safety and warning information, can be found in the Instructions for Use.

QUESTIONS?

If you or your healthcare professional have any questions or would like to know more about patient assistance, please don't hesitate to reach out to Akili Assist™ or call 1-844-AKILI-IQ (1-844-254-5447) Monday-Friday 9am-8pm ET.

EndeavorRx: Indications, Safety and Cautions

Indications for Use:

EndeavorRx is a digital therapeutic indicated to improve attention function as measured by computer-based testing in children ages 8-12 years old with primarily inattentive or combined-type ADHD, who have a demonstrated attention issue. Patients who engage with EndeavorRx demonstrate improvements in a digitally assessed measure, Test of Variables of Attention (TOVA), of sustained and selective attention and may not display benefits in typical behavioral symptoms, such as hyperactivity. EndeavorRx should be considered for use as part of a therapeutic program that may include clinician-directed therapy, medication, and/or educational programs, which further address symptoms of the disorder.

Cautions:

Rx only: Federal law restricts this device to sale by or on the order of a physician.

EndeavorRx should only be used by the patient for whom the prescription was written. For medical questions, please contact your child's healthcare provider. If you are experiencing a medical emergency, please dial 911.

EndeavorRx is not intended to be used as a stand-alone therapeutic and is not a substitution for your child's medication.

If your child experiences frustration, emotional reaction, dizziness, nausea, headache, eye-strain, or joint pain while playing EndeavorRx pause the treatment. If the problem persists contact your child's healthcare provider. If your child experiences a seizure stop the treatment and contact your child's healthcare provider.

EndeavorRx may not be appropriate for patients with photo-sensitive epilepsy, color blindness, or physical limitations that restrict use of a mobile device; parents should consult with their child's healthcare provider.

Please follow all of your mobile device manufacturer's instructions for the safe operation of your mobile device. For example, this may include appropriate volume settings, proper battery charging, not operating the device if damaged, and proper device disposal. Contact your mobile device manufacturer for any questions or concerns that pertain to your device.

Safety:

Of 538 participants using EndeavorRx, 50 participants (9.3%) experienced treatment-related adverse events (probable, likely), and three participants experienced treatment-related adverse events with the digital control, in studies where a control was used. EndeavorRx associated adverse events included frustration (6.1%), headache (1.3%), dizziness (0.6%), emotional reaction (0.4%), nausea (0.4%), and aggression (0.2%). There were no serious adverse events. All adverse events were generally transient. Only 3 events led to device discontinuation, and no subject reported lasting or irreversible effects after discontinuation.

